Finalidad

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- Easy on & off
- Removable touch fasteners
- Comfortable with cooling material

Juzo Night

Nighttime Compression Garment

Vêtement de compression nocturne

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Distributed by
Julius Zorn, Inc
3690 Zorn Drive
Cuyahoga Falls, OH 44223

Distributed by
Juzo Canada LTD.
www.juzo.ca

INNIGHT-1122





To learn more about Juzo medical
compression products go to;
juzousa.com or juzo.ca

© Juzo is a registered trademarks of Julius Zorn, Inc © 2023 Julius Zorn Incorporated

Please Recycle
Printed in USA

Freedom in Motion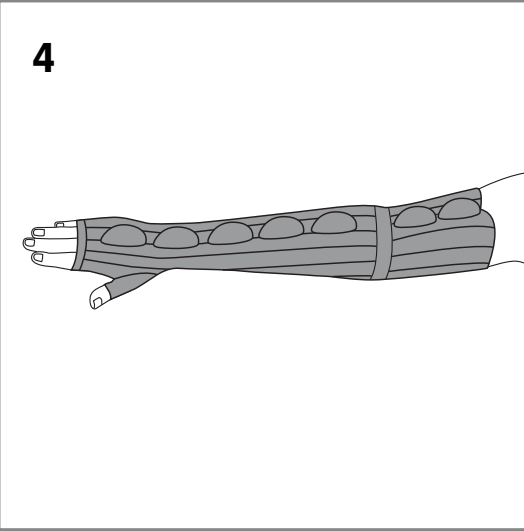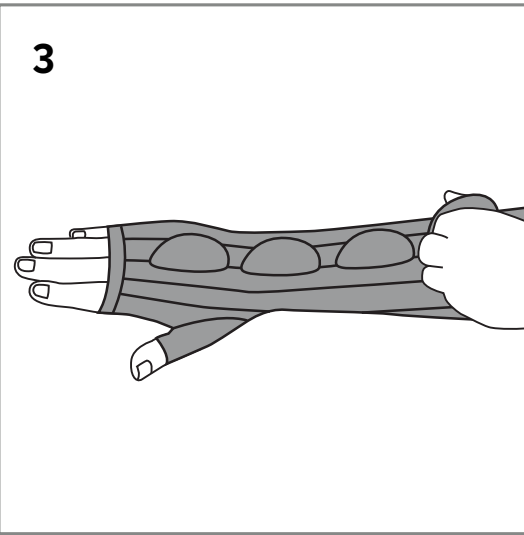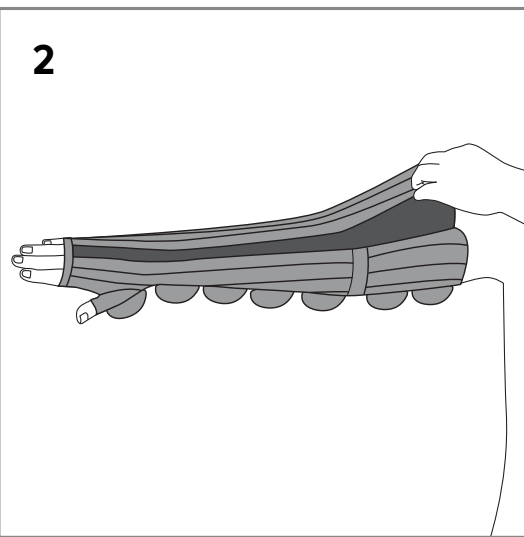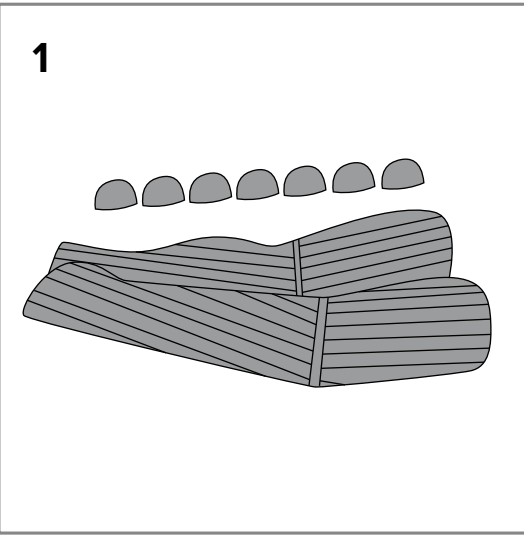

The Juzo Night effectively supports compression therapy, both at night and during daytime rest phases. Compression therapy reduces edema and accelerates venous return flow. Vertical foam channels form a unique surface structure that supports lymphatic drainage.

Notes for Use

A certified compression fitter or medical professional will be happy to show you how to use the product correctly. The hook and loop segments must be attached to the tabs before you wear the product for the first time. Make sure the label is on the inside. There must not be any large folds during donning or wearing. If required, you can increase the wrap’s compression by adjusting the tabs from the bottom to the top. The Juzo Night can also be used without the incorporated lining.

Arm Wraps

1. Juzo Night arm wraps are available in left and right versions. The touch fasteners must be attached to the product.

2. Put your arm into the integrated lining with your palm facing upwards. Now position the upper edge of the wrap below your armpit. The comfort zone of the arm wrap is located in the crook of the arm.

3. Bend your arm at a 90 degree angle and start by closing the tabs from the hand. Tighten them towards the outside of the hand.

4. Continue closing the other tabs from the bottom to the top.

5. Check that the product exerts pleasant, noticeable compression. If you feel pain or other problems, loosen the touch fasteners.

Please note

Only wear Juzo Night if it has been prescribed by a doctor. The products should only be dispensed by properly trained staff at a medical retailer. Further treatment success or a change in condition will require re-measurement and potentially a new product. Please contact your compression retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. The sewn-in textile label (if present) is important for the identification and traceability of the product. We therefore ask you not to remove the label under any circumstances.

Storage information and shelf life

Store in a dry place and protect from sunlight. Standard products have a maximum shelf life of 36 months. The products then have a usage period of 6 months. The usage period depends on correct handling (e.g. care, donning and doffing) and normal use of the products.

Washing and care instructions

Wash the Juzo Night after each use. Please note the care instructions on the textile label sewn into your product. Prior to washing, close all touch fasteners or, if this is not possible, remove them. The products can be machine-washed at 60°C using a laundry detergent for delicate fabrics. Tip: a mesh laundry bag additionally protects the product. Please do not use fabric softener. Never use bleach. Before drying, stretch the items while they are wet. You can reduce drying time by placing the Juzo Night on a thick towel, rolling the towel up tightly and firmly pressing out excess moisture. Do not leave the product in the towel and do not dry it on a radiator or in the sun. Juzo Night can be tumble-dried but must not be ironed.

Return Policy

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase.

Fabric

Details are provided on label sewn inside your compression garment.

Indications

Lymphedema, venous edema, lipedema, post-traumatic edema, post-operative edema

Absolute contraindications

Arterial insufficiency of upper extremity.

Relative contraindications

Decompensated heart failure (NYHA III + IV), septic phlebitis, acute bacterial, viral or allergic acute inflammation, swelling of the extremities for unknown reasons. Pronounced weeping dermatoses, intolerance to pressure or any of the components of the product, severe sensory disorders of the extremities, primary chronic polyarthritis, malignant lymphedema.

Important

Open wounds in the area to be treated must be covered with a sterile wound dressing adapted to exudate management before Juzo Night can be applied.

Side effects

Severe adverse events due to compression treatment such as skin necrosis, nerve damage or thromboembolic events are rarely encountered if compression is correctly used and contraindications are considered. Discomfort, dry skin and itching are the most frequently reported adverse events related to compression use. In the event any side effects occur from use of a garment, you should consult with a medical professional.

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