# Juzo

Think Green. Live Green.

Juzo is committed to reducing its carbon footprint for a greener planet.

Scan to view educational videos for compression products

**Juzo USA**  
3690 Zorn Drive  
Cuyahoga Falls, OH 44223

1 888-255-1300  
www.juzousa.com  
support@juzousa.com

**Juzo Canada Ltd.**  
1100 Burloak Dr. Ste. 300  
Burlington, ON L7L 6B2

1 888-255-1300  
www.juzo.ca  
support@juzo.ca

© Juzo, Slippie Gator, Keep Moving, Fiber Soft & Signature Collection are registered trademarks of Julius Zorn, Inc. Lycra is a registered trademark of Lycra Company. Tyvek is a registered trademark of Dupont.  
© 2022 Julius Zorn, Inc.

## Care & Application

### Compression Arm Sleeves

Freedom in Motion

#### Freedom to Make the Most of Every Day

Riding a bike. Working in the garden. Playing with kids. Taking a walk. Going to work. These are the keep moving benefits of Juzo compression garments, designed to improve your quality of life, not inhibit it. Family-owned since 1912, Juzo has led the compression therapy garment industry by creating high-quality products that offer you the freedom to maintain a more active lifestyle. Advancements like our FiberSoft technology, seasonal colors, attractive styles and fun print designs ensure you move in comfort without compromising fashion. Thank you for choosing Juzo.

#### Care and Maintenance

To maintain your Juzo arm sleeve, we recommend the following:

- Hand or machine wash warm daily to maximize therapeutic effectiveness and prolong the life of your Juzo garment.
- When machine washing and drying, use the lowest permanent press settings. A garment bag is strongly recommended.
- Wash dark colors separately from other clothing.
- Use Juzo washing detergent or another mild soap.
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives. These may damage your garment.
- Please refer to the washing symbols on the label sewn into your Juzo arm sleeve.
- Store your arm sleeve in a dry place at room temperature.

#### Return Policy

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase.

#### Fabric

Details are provided on label sewn inside your compression arm sleeve.

#### Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. As therapy progresses or new symptoms develop, it may be necessary to take new measurements for a new compression arm sleeve.

It is also recommended that you put your compression arm sleeve on first thing in the morning every day. Rough fingernails, jagged rings etc. can damage fabric; use a donning glove to protect the garment.

#### Indications

Lymphedema, venous congestion, post-traumatic swelling following burns and for scar treatment.

#### Absolute contraindications

Arterial insufficiency of upper extremity

#### Relative contraindications

Decompensated heart failure (NYHA III + IV), septic phlebitis, acute bacterial, viral or allergic acute inflammation, swelling of the extremities for unknown reasons. Pronounced weeping dermatoses, intolerance to pressure or any of the components of the product, severe sensory disorders of the extremities, primary chronic polyarthritis, malignant lymphedema.

#### Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied.


#### Side effects

Severe adverse events due to compression treatment such as skin necrosis, nerve damage or thromboembolic events are rarely encountered if compression is correctly used and contraindications are considered. Discomfort, dry skin and itching are the most frequently reported adverse events related to compression use. In the event any side effects occur from use of a garment, you should consult with a medical professional.


**Juzo Silver Garments should be removed prior to MRI Scans & Cardiac Defibrillations.**

**The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse.**


# Useful tips to help with donning your compression arm sleeve: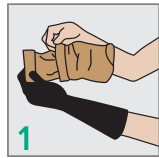


1 Fold the top half of the sleeve down over the bottom half of the sleeve.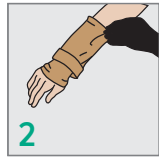

2 Using the folded edge, gently ease the sleeve over the hand and up the arm until the folded edge reaches the elbow. For sleeves with a shoulder strap, Make sure the shoulder attachment remains on the outer part of the arm.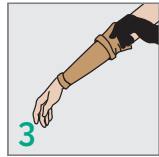

3 Using Juzo donning gloves, take the end of the top half of the sleeve and slide it slowly up the upper part of the arm towards the shoulder.

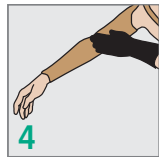4 Use the palm of your hand to equally distribute the knit and guide the sleeve into place. The knit should look even and fit smoothly over the length of the arm.

5 For sleeves with a shoulder strap, position one strap across the chest and the other around the back. Fasten the two ends together. Note: Should the strap be too tight or too loose, the length of the strap can be adjusted using the adjustment clip.

## Always use Juzo donning gloves when applying compression garments.

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Julus Zorn Inc. n'assume aucune responsabilité en raison d'une utilisation contraire à l'indication de ce produit.