JUZO COMPRESSION WRAP

Care & Application for Juzo Compression Wrap
Calf Segment

REVERSIBLE

beige or black garment

Freedom in Motion

Thank you for choosing Juzo, an international leader in medical compression garments since 1912.

Manufactured with care and detail, your Juzo Compression Wrap has several features that provide therapy, comfort, versatility and superior fit, including:

- Working pressure - Short stretch for a high working pressure.
- Gap resistant - Special notches and minimal overlying straps resist gapping and bulk.
- EZ pull tab - Aid for hassle free application.
- Reversible - Worn in either beige or black

Included In This Package:

- Juzo Compression Wrap
- Juzo Hook Fasteners
- Juzo Full Foot Wrap Liner (Full Foot Kit)
- Juzo Compression Cotton Stockinette (Open Toe Kit)
- Juzo Basic Compression Anklet in beige and black (Open Toe Kit)

Care and Maintenance

- Remove all the half circle hook fasteners from the straps
- Machine wash warm
- A garment bag is strongly recommended
- Tumble dry low
- Do not bleach
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives
- Store garments in a dry place at room temperature

Return Policy

Should it be necessary to return your garment for evaluation, Juzo requires that the garment be laundered (for hygienic reasons) prior to returning to the original place of purchase.

Fabric

Outer Shell - 88% Nylon, 12% Spandex / Inner Shell - 100% Neoprene

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. If you experience any discomfort, immediately remove the short-stretch compression wrap and consult with your medical professional.

- Juzo Compression Wrap is designed to be reversible. The half circle hook fasteners need to be attached to the garment on the opposing side of the color to be worn.
- Juzo Compression Cotton Stockinette option provides a sanitary layer between the garment and the skin and/or any wound dressing.
- Juzo Basic Compression Ankle option has mild compression designed to be worn with the wrap or by itself.

Indications

Primary & secondary varices, chronic venous insufficiency, post-thrombotic syndrome, venous stasis ulcers, pre and post-vein procedure and lymphatic edema.

Contraindications

Absolute: peripheral arterial disease, decompensated heart diseases, septic phlebitis, phlegmasia coerulea dolens.

Relative: Leg infection, intolerance to compression or garment fabric, sensory disturbance to the limb.

Cautions

No side effects are currently known. However, if a negative result should occur (i.e. skin irritations) from the use of our products, please discontinue use of the garment and consult with a medical professional. The manufacturer is not liable for damage /injuries which have been caused by improper handling or misuse.

Steps to Putting on the Calf Wrap

1. The Juzo Compression Wrap comes with a compression cotton stockinette and the Juzo Basic Anklet or the Juzo Full Foot Wrap Liner.
 - If using the Juzo Full Foot Wrap Liner, slide the liner over the foot and heel, pulling up the leg and over the calf. Position the top of the liner just below the knee, making sure the liner is smooth from top to bottom.
 - If using the compression cotton stockinette, place the garment covering the area from the ankle to just below the knee, with the cuffed edge closest to the ankle.
 - If required, the Juzo Basic Compression Anklet should be applied first, then slide the compression cotton stockinette over the ankle. This does not apply if you're using the Juzo Full Foot Wrap Liner.

Steps to Putting on the Calf Wrap

2. Attach the hook fasteners to the straps on the side opposite of the color to be worn. For example, if beige is the desired color, then the fasteners should be applied to the black side of the garment.

Steps to Putting on the Calf Wrap

4. While in a resting position, i.e. sitting down, position the wrap on the leg just below the knee. (2 fingers below the knee). The seam should run down the center of the backside of the leg.

4

Steps to Putting on the Calf Wrap

5. Secure the top strap of the wrap by counter pulling the top strap and the top EZ Pull at the same time. The strap should be just tight enough to gently hug the leg and hold the garment in place.

Steps to Putting on the Calf Wrap

6. Continue working down the leg and fasten the remaining straps using enough tension for the straps to gently hug the leg. We recommend simultaneously counter pulling the two adjacent straps.

Steps to Putting on the Calf Wrap

7. Once each strap is in place, start at the bottom of the leg and in-unison pull the bottom strap and EZ Pull in opposite directions.

7

Steps to Putting on the Calf Wrap

8. Continue working up the leg and counter pull the second and third straps. Make sure there are no gaps between straps. This can be best achieved by having adjacent straps slightly overlap and by inserting one strap into the notch.

8

Steps to Putting on the Calf Wrap

9. Pull the top strap and the EZ Pull until it is comfortably tightened.

Steps to Putting on the Calf Wrap

10. The wrap should have a snug fit but not be painfully tight. If you feel any pain or discomfort or the wrap feels too loose, then repeat steps 6 through 8, adjusting the level of tension that is applied.

Note: If you are using a Juzo Compression Wrap Foot Segment, see care and application directions for use.

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®Juzo, FiberSoft & Keep Moving are registered trademarks of Julius Zorn, Inc. © 2018 Julius Zorn Inc.
Julius Zorn, Inc. • 3690 Zorn Dr.
JUZO CANADA LTD.