Please take a few minutes to register your product online to enable us to provide you with the best customer service possible.

Benefits of Registration:

Notification

Allows us to contact you with important product notifications and updates.

Confirmation of Ownership

Provides us with a record

Improved Product Development

Help us continue to design quality products with your feedback.

Register your Juzo product online at:
www.juzoproducts.com

Juzo USA
P.O. Box 1088
Cuyahoga Falls, OH 44223
1 888-255-1300
www.juzousa.com
support@juzousa.com

Juzo Canada Ltd.
1100 Burloak Dr. Ste. 300
Burlington, ON L7L 6B2
1 888-255-1300
www.juzo.ca
support@juzo.ca

Freedom to Make the Most of Every Day

Riding a bike. Working in the garden. Playing with the kids. Taking a walk. Going to work. These are the keep moving benefits of Juzo compression garments, designed to improve your quality of life, not inhibit it. Family-owned since 1912, Juzo has led the compression therapy garment industry by creating high-quality products that offer you the freedom to maintain a more active lifestyle.

Advancements like our FiberSoft technology, seasonal colors, attractive styles and fun print designs ensure you move in comfort without compromising fashion. Thank you for choosing Juzo.

Care and Maintenance

To maintain your Juzo glove or gauntlet, we recommend the following:

- Hand or machine wash warm daily to maximize therapeutic effectiveness and prolong the life of your Juzo garment.
- When machine washing and drying, use the lowest permanent press settings. A garment bag is strongly recommended and available in the Juzo Compression Therapy Care accessory package.
- Wash dark colors separately from other clothing.
- Use Juzo washing detergent or another mild soap.
- DO NOT USE: bleach, chlorine, fabric softeners, or other laundry additives. These may damage your garment.
- Please refer to the washing symbols on the label sewn into your Juzo glove or gauntlet.
- Store your glove or gauntlet in a dry place at room temperature.

Return Policy

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase.

Details are provided on label sewn inside your compression glove & gauntlet.

Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. As therapy progresses or new symptoms develop, it may be necessary to take new measurements for a new compression glove or gauntlet.

It is also recommended that you put your compression glove or gauntlet on first thing in the morning every day. Rough fingernails, jagged rings etc. can damage fabric; use a Juzo donning glove to protect garment, on opposite hand.

Useful tips to help you with your compression glove or gauntlet.

1. Slip your hand straight into the glove or gauntlet
2. If necessary, adjust the fit of the glove. The seams between the fingers must be in direct contact with the skin. Check the fit of the glove in front of a light: if the fit is correct, there should be no space between your fingers and glove material.

Indications

Lymphedema and edema management, venous congestion, follow up treatment of burns and scars, post-traumatic edema.

Contraindications

Arterial insufficiency.

Cautions

Juzo Silver Garments should be removed prior to MRI Scans & Cardiac Defibrillations. In the event any side effects occur (i.e. skin irritation, etc.) from the use of a garment, you should consult with a medical professional.

The manufacturer is not liable for damage or injury caused by improper handling or misuse of its garments.

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